neurotech®

Kneehab®**XP**

**Distributed by:**

Neurotech NA Inc.  
A wholly-owned subsidiary of Theragen Inc.  
11220 Assett Loop, Suite 101  
Manassas VA 20109  
USA

Tel: 888-980-1197  
[www.neurotech.us](http://www.neurotech.us)

**Designed by & Manufactured for:**  
Theragen Inc., 11220 Assett Loop, Suite 101  
Manassas VA 20109, USA

Copyright © 2013 Theragen Inc. All rights reserved.

Caution: In the United States of America federal law restricts the device to sale or use by, or on the order of a physician or other practitioner licensed by the laws of the state in which he/she practices.  
neurotech® and Kneehab® XP are registered trademarks of Theragen Inc.

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Kneehab®**XP**

Kneehab+ Garment Instructional Photos

Figure A      Figure B      Figure C      Figure D      Figure E

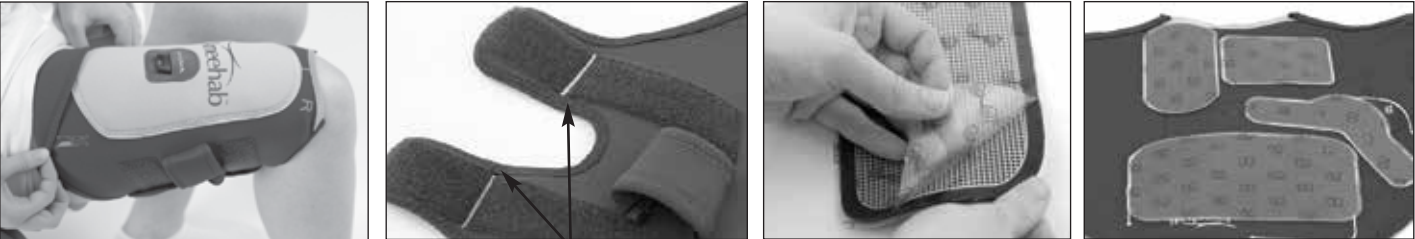
Figure F      Figure G - Line & Gap      Figure H      Figure I

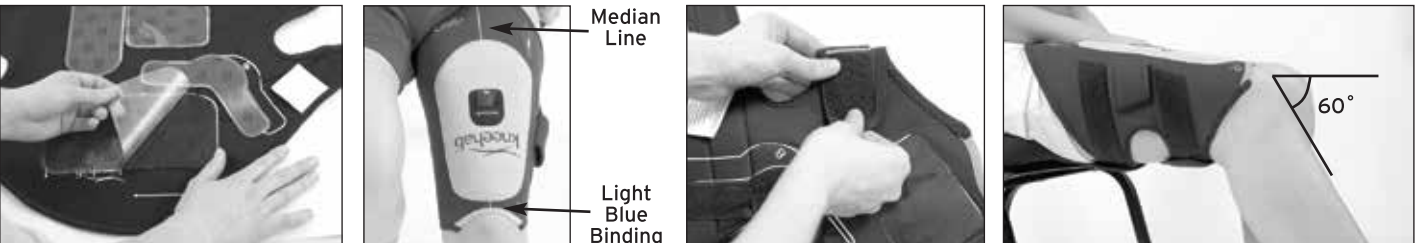
Figure J      Figure K      Figure L      Figure M

ENGLISH – INTRODUCTION & CONTACT DETAILS

Introduction

Dear Customer

Thank you for choosing the Kneehab® XP Conductive Garment from neurotech®. Kneehab XP was developed to facilitate the frequent and correct positioning and repositioning of large, multiple, and/or difficult to reach stimulation sites associated with the quadriceps area of the leg.

The pack contains your Kneehab XP conductive garment, conductive gel pads and a user manual.

The conductive gel pads need to be replaced periodically and are readily available by contacting your local Neurotech supplier.

If you have further questions regarding Kneehab XP, please contact your prescribing physician, the Neurotech distributor that provided you with Kneehab XP or contact your Neurotech team at the address/phone number below.

Yours sincerely,  
The Neurotech Team.

If you have questions or require further information please contact:

**Neurotech NA Inc.**  
A wholly-owned subsidiary of Theragen Inc.  
11220 Assett Loop, Suite 101  
Manassas VA 20109  
USA  
Tel: 888-980-1197  
[www.neurotech.us](http://www.neurotech.us)

# TABLE OF CONTENTS

---

<b>1.0</b>	<b>Safety Information</b>	<b>4</b>
<b>2.0</b>	<b>Contraindications</b>	<b>4</b>
2.1	Warnings	5
2.2	Precautions	5
<b>3.0</b>	<b>Frequently Asked Questions</b>	<b>6</b>
<b>4.0</b>	<b>Getting to Know Kneehab XP</b>	<b>7</b>
4.1	Kneehab XP Pack Contents	7
4.2	Description of Garment	7
<b>5.0</b>	<b>Setting-Up a Kneehab XP Treatment</b>	<b>8</b>
5.1	Step-By-Step Treatment Guide	10
<b>6.0</b>	<b>System Maintenance</b>	<b>11</b>
<b>7.0</b>	<b>Technical Specifications &amp; Accessories</b>	<b>12</b>
<b>8.0</b>	<b>Warranty</b>	<b>13</b>

## 1.0 Safety Information

### Validity

The information and technical data contained in this document relates to the Kneehab XP conductive garment provided with this User Manual. The information and technical data disclosed in this document are proprietary to Theragen Inc. (Theragen) and may only be used and disseminated for the purposes and to the extent specifically authorized in writing by the company.

### Disclaimers

All items of equipment manufactured and sold by Neurotech are rigorously checked and tested prior to shipment. However, the use of this equipment is outside of the control of the company. Neurotech only accepts responsibility for the safety, reliability and performance of the equipment when it is operated in accordance with the instructions herein and within the given specifications. Therefore, the user must bear full responsibility for any actions arising out of the use or misuse of this equipment. Any modifications, repairs or servicing must be undertaken by authorized Neurotech personnel.

Kneehab XP is designed by and manufactured for Neurotech. Copyright © 2013 by Theragen Inc., all rights reserved. neurotech® is a registered trademark of Theragen Inc.

The sale and/or operation of this equipment is subject to legislation in a number of localities. Compliance with this legislation rests with the importer, dealer, or user of the equipment as appropriate.

### Prescription Required:

Caution: In the United States of America federal law restricts the device to sale or use by, or on the order of, a physician or other practitioner licensed by the laws of the state in which he/she practices.

### Intended Use: Kneehab XP Conductive Garment

The Kneehab XP Conductive Garment and associated accessories are indicated for use with the Neurotech Kneehab XP controller to facilitate the frequent and correct positioning and repositioning of large, multiple, and/or difficult to reach stimulation sites associated with the quadriceps area of the leg.

## 2.0 Contraindications Warnings & Precautions

### Contraindications

This device must not be used on persons with cardiac pacemakers, defibrillators, or other implanted metallic or electronic devices.

**Warnings**

- The long term effects of prolonged use of electrical stimulation are unknown;
- This garment should not be applied to any part of the body except for the quadriceps (thigh) as instructed. Stimulation applied over the neck, across the chest or head may cause an adverse reaction;
- Conductive gel pads should be applied only to normal, intact, clean skin. Conductive gel pads should not be applied over open wounds or over swollen, infected, or inflamed areas or skin eruptions, e.g., phlebitis, thrombophlebitis, varicose veins, etc;
- Conductive gel pads should not be shared with other persons. Each person should have their own set of conductive gel pads; otherwise, undesirable skin reactions may occur;
- Self-adhesive conductive gel pads should be replaced if they no longer stick firmly to the skin; and
- Stimulation should not be applied when in the bath or shower.

**Precautions**

- Some persons may experience skin irritation or hypersensitivity due to the electrical stimulation or electrical conductive medium (gel).
- This device should be kept out of the reach of children; and
- The device should be used only with the conductive gel pads, and accessories recommended for use by the manufacturer.

**Adverse Reactions**

- Skin irritation and burns beneath the conductive gel pads have been reported with the use of stimulation conductive gel pads applied to the skin.

**Do not use Kneehab XP if:**

- Positioning the conductive gel pads necessitates placement over areas at which drugs/ medicines are administered by injection (short term or long term) e.g. Hormone treatment.

**Note:**

- For hygiene reasons, the Conductive Garment and conductive gel pads are for single-person use only.

**Important**

- There is a separate Kneehab XP garment for use with the right or left leg. Under no circumstances should the left garment be used on the right leg or vice versa. A letter printed on every garment (L or R) indicates which leg that garment is designed for i.e. L: left, R: right;
- Position your Kneehab XP only as outlined in this User Manual. Ensure the light-blue binding at the narrow end of the garment is placed just above the kneecap;
- Do not adjust the Kneehab XP garment when it is connected to an electrical stimulation device which has been switched on; pause the device, make the adjustment then recommence treatment; and
- The safety information provided in this User Manual must be followed.
- Adult supervision is required if a child is prescribed this product.

### 3.0 Frequently Asked Questions

**Q: What is Kneehab XP?**

A: The Kneehab XP Conductive Garment is an innovative, garment-based solution, specifically designed to facilitate the frequent and correct positioning and repositioning of multiple conductive gel pads for the treatment of the quadriceps area of the leg.

**Q: How often should I use Kneehab XP?**

A: Your prescribing physician will advise you on the duration of your treatment program.

**Q: Can Kneehab XP be used in conjunction with other forms of therapy?**

A: Kneehab XP can be used as an adjunct to most forms of therapy for knee injuries. See page 2 of this manual for Contraindications (situations when Kneehab XP should not be used).

**Q: How often should I replace the conductive gel pads?**

A: This depends on a number of factors including skin type, storage conditions and regularity of use. With proper care, as per this manual and typical use, the gel pads have an estimated usage of approximately one month. Please store the conductive gel pads in accordance with the instructions on page 11 of the User Manual.

**Q: How do I position Kneehab XP on the leg?**

A: See pages 8-10 of this User Manual for full instructions on positioning Kneehab XP on the leg.

**Q: Can Kneehab XP be used on other body parts?**

A: No. Kneehab XP should only be used as directed in this User Manual and not applied to any other body parts.

**Q: How should I care for my Kneehab XP?**

A: See page 11 of the User Manual for full instructions on caring for your Kneehab XP Conductive Garment.

**Q: Who should I contact if I have additional queries?**

A: Contact your prescribing clinician or your Neurotech team if you have any questions regarding usage of your Kneehab XP Conductive Garment.

**Q: What should I do if I experience an adverse reaction?**

A: If any irritation, skin reaction, hypersensitivity or other adverse reaction is experienced, discontinue use and contact your prescribing clinician. A slight reddening of the skin is normal under the conductive gel pads during, and for a short time after, treatment. This should disappear within approximately 30 minutes.

## 4.0 Getting to Know Kneehab XP Conductive Garment System

Kneehab XP is an easy to use, conductive garment-based solution, specifically designed for home use.

### 4.1 Kneehab XP Pack Contents

#### 1. Kneehab XP Conductive Garment:

Comfortable garment, which fastens around your thigh, above the kneecap, for quick and easy use. It contains a connector socket, into which the Controller is plugged. Please ensure you have the correct garment (Left/ Right leg) before using the Kneehab XP Conductive Garment.

#### 2. Kneehab XP Conductive Gel Pads:

The conductive gel pads are placed on the garment to conduct signals from the Controller to the quadriceps muscles.

#### 3. Kneehab XP User Manual:

The Kneehab XP User Manual provides a guide to product use and safety information.

### 4.2 Description of the Kneehab XP Garment

#### Important:


Kneehab XP garments are supplied in 2 different sizes - **Small (S)** and **Medium-Large (M-L)**. To ensure you get the full benefit of a Kneehab XP treatment, you **must** use the conductive gel pads designed to fit your specific garment. Before opening your conductive gel pad pack, please check that you have the correct conductive gel pads:

- If you have the **Small** garment, the conductive gel pad pack will have a circled '**S**' on the front and will identify that the conductive gel pads are **Type: 618** on the back of the pack.
- If you have the **Medium-Large** garment, the conductive gel pad pack will have a circled '**M-L**' on the front and will identify that the conductive gel pads are **Types: 613, 614, 615, 616** on the back of the pack.

The inside of the garment has been printed with conductive gel pads placement guides, showing you exactly where to place the 4 conductive gel pads. Two of the locations will vary depending on your size (**A/a** and **B/b**).

For example, the correct position for the large conductive gel pads if you are tall is marked '**A**', whereas a smaller person would place the same conductive gel pad within the area marked '**a**'.

See pages 9-10 for full details on how to determine your correct conductive gel pads placement.

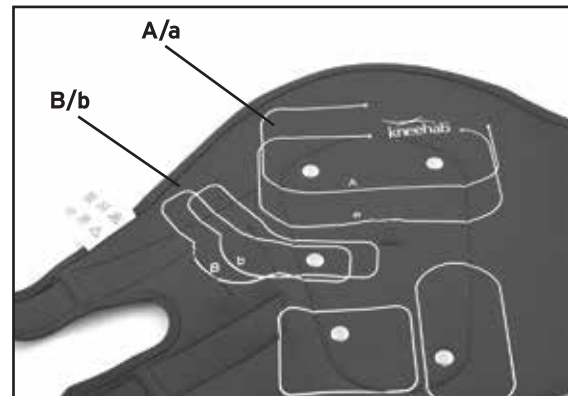## 5.0 Setting-Up A KneeHab XP Treatment

### Step 1 - Correct Conductive Garment confirmation

#### Important:

The correct garment **MUST** be used for the leg being treated. L is printed on the garment designed specifically for the left leg. The printed figure of a human also shows the garment positioned on the left leg (Figure A). R is printed on the garment designed specifically for the right leg. The printed figure of a human also shows the garment positioned on the right leg (Figure B).

Please check now to ensure you have the correct garment for the leg to be treated. If the garment you have is not for the correct leg, please contact your prescribing physician or the Neurotech distributor that provided you with KneeHab XP.

### Step 2 - Initial garment fitting

- Sit down with your leg as straight as possible. For the best fit and most effective treatment it is important to have your leg as straight as possible during garment fitting.
- Place the garment on your leg so that the light blue binding lies just above the kneecap (approx. 0.5cm/ 0.25") and the printed median line lies along the center of your thigh (Figure C).
- Fasten both pairs of straps (Figure D). For the best fit, fasten the pair of straps closest to the kneecap first. When the straps are fastened the garment should have a comfortable yet firm fit.

### Step 3 - Sizing determination

With the garment still fastened to your leg, sit upright with your knee bent at 90° (Figure E).

#### To determine length sizes:

- If the whole garment lies flat along the length of your thigh (Figure E), you are a tall fit and must place the large rectangular conductive gel pad within the area marked 'A'.
- If you have to fold down the top of the garment at the groin (Figure F), you are a small fit and must place the large rectangular conductive gel pad within the area marked 'a'.

#### To determine width sizes:

There is a line printed on the garment near one end of the straps to help you determine whether you are a wide or a narrow fit. If you have difficulty seeing this line, there is also a gap in the fastening material to help you (Figure G). While the garment is still fastened to your leg, sit upright with your knee bent at 90° (Figure E).

- If the strap does not overlap the printed line, or if you can still feel the gap, you are a wide fit and must place the L-shaped conductive gel pad within the area marked 'B'.
- If the strap does overlap the printed line, or if you cannot feel the gap, you are a narrow fit and must place the L-shaped conductive gel pad within the area marked 'b'.

### Step 4 - Adhesive Conductive Gel Pad Placement

- Remove the garment from your thigh and place it on a flat surface with the white conductive gel pad placement guides facing you.
- Kneehab XP is supplied with conductive gel pads. Remove these from their plastic bag.
- Remove the liners from the **patterned** side of the conductive gel pads (Figure H). Keep these liners for later.
- Stick the conductive gel pads onto the garment within their respective printed outlines.
- Do not remove the liners from the **black** sides of the conductive gel pads yet.

#### Important:

- Place the largest conductive gel pad within outline "A" or "a" depending on your sizing, determined in Step 3.
- Place the L-shaped conductive gel pad within outline "B" or "b" depending on your sizing, determined in Step 3.
- The remaining two conductive gel pads should be placed within their respective matching outlines (Figure I).
- Ensure that all conductive gel pads lie perfectly within the conductive gel pad placement guides on the garment.
- Ensure the metal studs are completely covered by all of the conductive gel pads.
- Press the edges of each conductive gel pad firmly onto the garment to ensure good adhesion.
- Replacement conductive gel pads are available from the distributor that provided you with this product or Neurotech.

**Your Kneehab XP is now set up and ready for connection to the Kneehab XP Controller.**

## 5.1 Step-By-Step Treatment Guide

### Step 1 - Garment Fitting

Using mild soap and water, cleanse the area where you will be placing the garment. The conductive gel pads do not adhere well to dirt, oil, creams or other cosmetics on the skin. Ensure the skin is dry.

Remove the liners from the **black** side of each conductive gel pad (Figure J). Do not dispose of these liners, as they will need to be placed back onto the conductive gel pads between treatment sessions.

Sit down with your leg as straight as possible. For the best fit and most effective treatment it is important to have your leg as straight as possible during garment fitting.

Place the garment on your leg so that the light blue binding lies just above the kneecap (approx. 0.5cm/ 0.25") and the printed median line lies along the center of your thigh (Figure K).

Fasten both pairs of straps. For the best fit, fasten the pair of straps closest to the kneecap first. When the straps are fastened the garment should have a comfortable yet firm fit.

If you are a very narrow fit, the straps may overlap some of the conductive gel pads. However, for an effective and safe treatment, it is important that the whole surface of all four conductive gel pads are in contact with the skin. You can eliminate the overlapping problem by folding the straps inwards at the printed line before fastening (Figure L).

### Important Note:

Caution is required when using Kneehab XP because the strong quadriceps contraction it elicits may cause involuntary lower leg movement. If not otherwise directed by your prescribing physician we recommend using Kneehab XP while sitting with your leg bent at an angle between 0-60 degrees (Figure M) and your foot secured firmly to prevent lower leg movement.

## 6.0 System Maintenance & Storage

### Caring for your Kneehab XP - Washing Instructions:

Kneehab XP is washable but you must first remove the Controller and conductive gel pads. Replace the liners on the **patterned** side of each conductive gel pad until you are ready to use your Kneehab XP again. You must always follow the instructions on the label when washing the garment.

Never machine wash your Kneehab XP garment. Hand wash in lukewarm water and NEVER wring to remove water. This protects the internal composition of the garment from damage.

Do not use any bleach when washing your Kneehab XP garment.

Do not dry clean your Kneehab XP garment.

Do not tumble dry your Kneehab XP garment. The garment should be dried on a flat surface. Do not dry over anything hot (e.g. a radiator or any other direct heat source). Ensure the garment is completely dry before use. The straps should be fastened prior to washing.

The Kneehab XP garment should not be ironed.

### Storage

Remove the garment by carefully peeling the conductive gel pads off your leg. Ensure that the conductive gel pads remain attached to the garment. Place the garment on a flat surface with the conductive gel pads facing upward and replace the liners on the **black** side of each one. If the conductive gel pads have lifted from the garment, press the edges back into position. Close the straps on the garment, ensuring that the conductive gel pads all remain flat. Store the garment, with the conductive gel pads still attached, on a flat surface in a cool dry place.

### Repair, Service & Modification

Access to the interior is not required for maintenance purposes.

If your Kneehab XP garment is damaged, you should not use it, but should return it to the distributor that provided you with this product or to Neurotech for repair or replacement. Repairs, service and modifications may not be carried out by anyone other than qualified service personnel authorized by Neurotech. Neurotech will not accept any responsibility if these guidelines and instructions are not followed.

## 7.0 Technical Information & Accessories

### **Kneehab XP Conductive Garments:**

**Right Leg, Small** (1274-3332)

**Left Leg, Small** (1274-3333)

**Right Leg, Medium/Large** (1274-3324)

**Left Leg, Medium/Large** (1274-3325)

### **Kneehab XP Adhesive Conductive Gel Pads:**

**Gel Pads for Small Kneehab:** Type: 618

**Gel Pads for Medium/Large Kneehab:** Type: 613/ 614/ 615/ 616

Kneehab XP uses conductive gel pads which have been specifically designed for this unit. Replacement conductive gel pads are available. To order replacement conductive gel pads, please contact the Neurotech or the Neurotech distributor that provided you with Kneehab XP. Only conductive gel pads specified by Theragen Inc. for use with Kneehab XP may be used. All adhesive conductive gel pads are provided with their own instructions where necessary, including cleaning, storage and care instructions. With proper care, as per this manual and typical use, the gel pads have an estimated usage of approximately one month.

### **Disposal of Conductive Gel Pads**

Used conductive gel pads must never be disposed of in a fire, but in accordance with your national or state laws governing the disposal of such items. Return all other contents to Theragen, Neurotech or the distributor who provided you with Kneehab XP.

**Intended use:** Conductive Garment

### **Environmental Specifications:**

**Operating Range:** Temperature: 10 to 35°C / 55 to 95°F  
Humidity: 20 to 65% RH

**Transport & Storage Range:**  
Temperature: 0 to 55°C / 32 to 131°F  
Humidity: 10 to 90% RH

## Disposal of device

At the end of the product lifecycle, do not throw out this product with the normal household garbage, but bring it to a collection point for the recycling of electronic equipment.

Some product materials can be re-used if you bring them to a recycling point. By re-using some parts or raw materials from used products you can make an important contribution to the protection of the environment. Please contact your local authorities if you need more information about collection points in your area.

Waste Electrical and Electronic Equipment can have potentially harmful effects on the environment. Incorrect disposal can cause harmful toxins to build up in the air, water and soil and can be harmful to human health.

## Garment Material:

**Outer Fabric:** 100% Nylon

**Inner Fabric:** 70% Polychloroprene, 30% Polyurethane

**Binding:** 82% Nylon, 18% Elastane

**Fastenings:** 100% Nylon

## 8.0 Warranty

Neurotech offers a warranty of 3 years for the Kneehab XP Conductive Garment\*. Should your Kneehab XP develop a fault within the specified time, Neurotech will replace or repair your Kneehab XP garment free of charge, provided that it:

- has been used for its intended purpose and in the manner described in this instruction manual.
- has not been connected to an unsuitable power source.
- has not been subjected to misuse or neglect; and
- has not been modified or repaired by anyone other than an approved Neurotech distributor or agent.

This warranty complements existing national guarantee obligations and does not affect your statutory rights as a consumer.

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Kneehab XP est conçu et fabriqué par Neurotech. Copyright ©2013 par Theragen Inc., tous droits réservés. neurotech® est marque de commerce déposée de Theragen Inc.