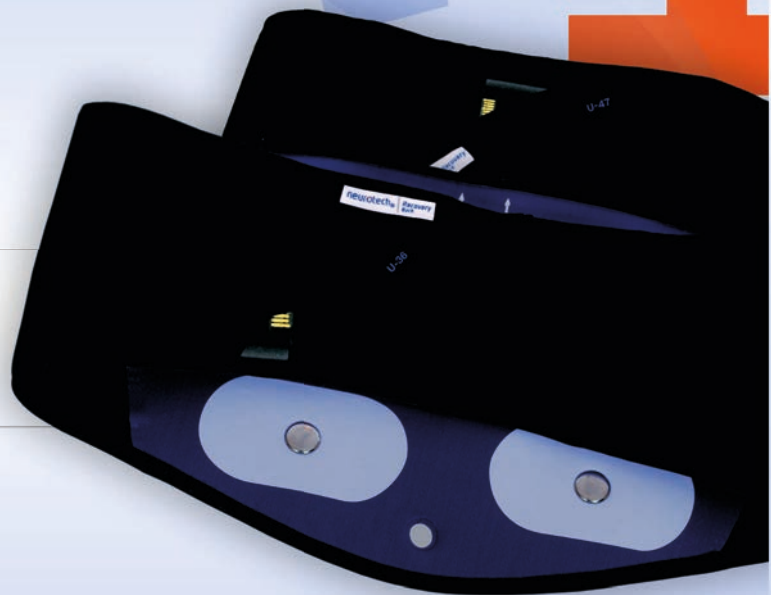
neurotech®
embrace life.

Neurotech® Recovery - Back

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Introduction

Dear Customer

Thank you for selecting the Neurotech Recovery - Back from Neurotech®. These conductive garments are designed to help ensure proper conductive gel pad placement on the low back and abdominal muscles. The following provides instructions for use of the conductive garments. Also, included is an overview for the garments' operation with a Transcutaneous Electrical Nerve Stimulation (TENS) or Neuromuscular Electrical Stimulation Controller (Neurotech® Plus). For further information, refer to the TENS or NMES Controller instruction manuals.

Please ensure that you read all of the instructions carefully before using Neurotech Recovery - Back. If you have further questions regarding these garments, please contact your health care professional, distributor that provided you with the Neurotech Recovery - Back, or Neurotech at the address/phone number below.

If you have questions or require further information please contact:

neurotech®

A Division of Bio-Medical Research Ltd.

PO Box 5179

Hoboken, NJ 07030

Tel: 800-901-5667

Web: www.neurotech.us

TABLE OF CONTENTS

1.0	Safety Information	4
2.0	Contraindications, Warnings and Precautions	5
3.0	Frequently Asked Questions	6
4.0	Getting to Know Neurotech Recovery - Back Garments	8
5.0	Setting Up Conductive Garments	9
5.1	Gel Pads on Conductive Garments	9
5.2	Setting up a Back Treatment	11
5.3	Setting up an Abdomen Treatment	11
6.0	Using Garments with the Neurotech Plus for a Treatment	12
7.0	System Maintenance & Storage	14
8.0	Technical Information & Accessories	15
9.0	Warranty & Program Information	17

1.0 Safety Information

Validity

The information and technical data contained in this document relates to the Neurotech Recovery - Back conductive garment system with this manual. The information and technical data disclosed in this document are proprietary to Neurotech and may be used and disseminated only for the purposes and to the extent specifically authorized in writing by the company.

Disclaimers

All equipment manufactured and sold by Neurotech, a Division of Bio-Medical Research Ltd., are rigorously checked and tested prior to shipment. However, the use of this equipment is outside of the control of the company. Neurotech only accepts responsibility for the safety, reliability and performance of the equipment when it is operated in accordance with the instructions herein and within the given specifications. Therefore, the user must bear full responsibility for any actions arising out of the use or misuse of this equipment. Any modifications, repairs or servicing must be undertaken by authorized Neurotech personnel.

Restrictions

The sale and/or operation of this equipment is subject to law in the various states/countries. Compliance with this legislation rests with the importer, dealer, or user of the equipment as appropriate.

Indications For Use - Neurotech Recovery - Back

The Neurotech Recovery - Back conductive garment for the back and associated accessories are indicated for use with Neurotech Stimulators to facilitate the frequent and correct positioning and repositioning of large, multiple, and/or difficult to reach stimulation sites associated with the low back.

The Neurotech Recovery - Back conductive garment for the abdomen and associated accessories are indicated for use with Neurotech Stimulators to facilitate the frequent and correct positioning and repositioning of large, multiple, and/or difficult to reach stimulation sites associated with the abdominal area.

Prescription Required:

Caution: In the United States of America federal law restricts the device to sale or use by, or on the order of, a physician or other practitioner licensed by the laws of the state in which he/she practices.

2.0 Contraindications, Warnings and Precautions

Side effects harmful to health have not been reported to date. Please consult your health care professional if you have any concerns.

Contraindications

- This device must not be used on persons with cardiac pacemakers, defibrillators, or other implanted metallic or electronic devices.

Warnings

- The long term effects of prolonged use of electrical stimulation are unknown;
- This garment should not be applied to any part of the body except for the low back or abdomen muscles as instructed;
- Stimulation applied over the neck, across the chest or head may cause an adverse reaction;
- Conductive gel pads should be applied only to normal, intact, clean skin. Conductive gel pads should not be applied over open wounds or over swollen, infected, or inflamed areas or skin eruptions, e.g., phlebitis, thrombophlebitis, varicose veins, etc.
- Conductive gel pads should not be shared with other persons. Each person should have their own set of conductive gel pads, otherwise, undesirable skin reactions may occur;
- Self-adhesive conductive gel pads should be replaced if they no longer stick firmly to the skin; and,
- Stimulation should not be applied when in the bath or shower.

Precautions

- Some persons may experience skin irritation or hypersensitivity due to the electrical stimulation or electrical conductive medium (gel);
- This device should be kept out of the reach of children; and,
- The device should be used only with the conductive gel pads, and accessories recommended for use by the manufacturer.

Adverse Reactions

Skin irritation and burns beneath the conductive gel pads (gel pads) have been reported with the use of stimulation gel pads applied to the skin.

Do not use Neurotech Recovery - Back if: Positioning the gel pads necessitates placement over areas at which drugs/medicines are administered by injection (short term or long term) e.g. hormone treatment.

Note: For hygiene reasons, the conductive garments and gel pads are for single-person use only.

3.0 Frequently Asked Questions

Q: What is Neurotech Recovery - Back?

A: The Neurotech Recovery - Back conductive garments are an innovative, garment-based solution specifically designed to facilitate the frequent and correct positioning and repositioning of multiple conductive gel pads for the treatment of the low back and abdomen muscles.

Q: How often should I use Neurotech Recovery - Back garments?

A: Your prescribing physician will advise you on the duration of your treatment program.

Q: Can Neurotech Recovery - Back be used in conjunction with other forms of therapy?

A: Neurotech Recovery - Back can be used as an adjunct to most forms of therapy for low back injuries. See page 5 of this manual for contraindications (situations when Neurotech Recovery - Back should not be used).

Q: How often should I replace the gel pads?

A: This depends on a number of factors including skin type, storage conditions and regularity of use. With proper care, as per the manual, gel pads have an estimated lifespan of 1 month. Please store the gel pads in accordance with the instructions on page 15 of the User Manual.

Q: How do I position Neurotech Recovery - Back on the low back?

A: See page 11 of this User Manual for full instructions on positioning for low back and abdominal muscles.

Q: Can I use the abdomen garment for post natal exercises?

A: Yes, but you must wait a minimum of 6 weeks after childbirth before you begin using and you must consult your physician first. If you've had a caesarean in the past 3 months, consult your physician for approval before using the garment.

Q: Can Neurotech Recovery - Back garments be used on other body parts?

A: No. Neurotech Recovery - Back garments should only be used as directed in this User Manual and not applied to any other body parts.

Q: How should I care for my Neurotech Recovery - Back garments?

A: See page 14 of the User Manual for full instructions on caring for your Neurotech Recovery - Back garment.

Q: Who should I contact if I have additional questions?

A: Contact your prescribing clinician, the distributor who provided you with the Neurotech Recovery - Back or your Neurotech team if you have any questions regarding usage of Neurotech Recovery - Back conductive garments.

Q: What should I do if I experience an adverse reaction?

A: If any irritation, skin reaction, hypersensitivity or other adverse reaction is experienced, discontinue use and contact your prescribing clinician. A slight reddening of the skin is normal under the conductive gel pads during and for a short time after treatment. This should disappear within approximately 30 minutes.

Q: The stimulation is uncomfortable. How can I improve this?

A: Make sure the gel pads are positioned correctly and that they are pressed firmly against your skin. Switch off your Controller and reposition the Neurotech Recovery - Back if necessary. Ensure the metal studs are fully covered by the gel pads. Ensure the gel pads don't need to be changed.

Q: I can feel a tingling sensation in my legs during use of the abdomen garment. What should I do?

A: This indicates that the gel pads are over your hip-bones. Moving the two smaller gel pads upwards and inwards on your waist (i.e. towards the center of your body) should prevent this. Remember to pause or switch off the Controller before adjusting the conductive garment or gel pads.

Q: When using the Neurotech Recovery - Back Abdomen conductive garment, I can feel my waist muscles exercising but not my stomach muscles. What should I do?

A: Pause the program and reposition the central gel pad slightly lower on your stomach. If this doesn't help, move the 2 small gel pads to a smaller figure setting on the conductive garment (towards the central gel pad).

Q: When using the Neurotech Recovery - Back Abdomen conductive garment, I can feel my stomach muscles exercising but not my waist muscles. What should I do?

A: Pause the program and reposition the central gel pads slightly higher on your stomach. If this doesn't help, move the 2 smaller gel pads to a larger figure setting on the conductive garment (away from the central gel pad).

4.0 Getting to Know Neurotech Recovery - Back Garments

Neurotech Recovery - Back conductive garments are easy to use and specifically designed for home use.

4.1 Neurotech Recovery – Back Garment Pack Contents

1. Neurotech Recovery - Back conductive garment

This conductive garment fastens around your low back, for quick and easy use. It contains a connector socket, into which the Controller is plugged.

2. Neurotech Recovery - Back Abdomen conductive garment

Universally sized garment, which fastens around your abdomen. It contains a connector socket, into which the Controller is plugged. This garment cannot be used while using the back garment. Only one garment can be used at a time.

3. Gel Pads

The gel pads are placed on the garments to conduct signals from the Controller to the low back or abdominal muscles. There is a set of gel pads for use on the back garment and a set for use on the abdomen garment.

4. User Manual

The Neurotech Recovery - Back User Manual provides a guide to product use and safety information.

5. Extender Strap

Used to increase length of garments (not shown).

Note: Storage bag also included (not shown).

4.2 Description of the Neurotech Recovery - Back Garment

The Neurotech Recovery - Back product includes a garment for use on your back and a conductive garment for use on your abdominal muscles. The garments are designed to help with the correct and repeatable positioning of gel pads while you treat your back or abdominal muscles. Once you apply the gel pads to the garment, you can leave them adhered for future use or until the gel pads need replacing.

Neurotech Recovery - Back conductive garment

- Small (25" - 34")
- Medium / Large (34" - 45") with an extender strap for the back garment to allow up to 56" waist fit.

Neurotech Recovery - Back Abdomen conductive garment

- One size fits all for the Abs belt up to a waist circumference of 47". The extender strap can be used to lengthen the belt to 56".
- To allow for different body shapes, it is recommended that the larger circumference of either the hips or waist should be used when selecting the appropriate size.
- Please refer to a trained specialist for correct fitting of your garment.

5.0 Setting Up Conductive Garments

This section explains how to apply the back garment when using the conductive garments for the first time. Gel pads are required for use of both the back and abdomen conductive garments. Do not operate these garments without gel pads.

5.1 Gel Pads on Conductive Garments

Back Garment

The inside of the garment has been printed with gel pad placement guides, showing you exactly where to place the four pads.

Step 1: Lay out the Neurotech Recovery - Back on a flat surface with the blue panel with white markings facing you. (Fig. A)

Note: Your Recovery Back is packaged with two different types of gel pads. Be sure to use the oval shaped pads as they are designed specifically for use on the back.

Step 3: Place the four gel pads, grid patterned side down, firmly on the garment within outlined areas. (Fig. C)

Step 4: Remove the liners from the black side of all four gel pads. Do not throw these liners away as you will need to put them back on to the gel pads at the end of your session.

Abdomen Garment

This section explains how to set-up the gel pads on the abdomen garment before using it for the first time.

Step 1: Turn the abdomen garment over so the 3 silver studs are facing upwards. Remove the three gel pads from their package. There is one large square pad and two smaller oval pads. One side of each pad has a grid pattern on it while the other side is plain back. See below for placement of gel pads.

Step 2: Remove the liners from the **grid patterned side** of the large, square gel pad and place it grid side down over the middle stud and between the white lines. (Fig. D)

Step 3: Now, remove the liners from the **grid patterned side** of the two smaller oval shaped gel pads and place them on either side of the large pad on the garment.

Note: One size fits all for the abdomen garment however, the location of the smaller side pads depends on your waist size. In the example below, the smaller pads are placed nearest to the large, central pad. This is for a user whose waist size is 24"-32". Use the sizing guidelines on the right, to help you choose your correct pad location.

Abdomen Garment Gel Pad Locations: Place a gel pad on each side of the center pad at designated locations.

1. For waist sizes 40" - 47" (106-119cm)
2. For waist sizes 32" - 40" (81 - 106cm)
3. For waist sizes 24" - 32" (61 - 81cm)

Important: Before you progress, make sure that all silver studs are completely covered by the gel pads. Press the edges of all three pads firmly onto the garment before use.

Extender Strap: Use of the extender strap will be required for waist sizes from 47" to 56". Use the outer gel pad locations when using the garment extender strap. Attach the extender strap to one end of the abdomen garment in order to extend the overall length.

Step 4: Remove the liners from the black side of all three pads. Do not throw these liners away as you will need to put them back on to the pads at the end of your session.

5.2 Setting Up A Back Treatment

This section explains how to apply the back garment before using it for the first time. Gel pads should be applied to the garment before putting it on.

Back Garment Placement

Step 1: The Neurotech Recovery - Back should always be centered over the spine. Place the peak of the curved section (pointing down) directly against your spine to align the garment and then hold the ends in both hands as shown. (Fig. E). Ensure the garment is spread equally on both sides of the body.

You can check the horizontal positioning by putting your finger into the die-cut circle on the back of the garment and feel for your spine. The die-cut hole should be centered low and over your spine. (Fig. F)

Step 2: Place your fingers inside the pockets at either end of the garment and pull the two ends around to meet in front of you. Wrap the left-hand side around your body first. (Fig. G)

Step 3. Next, wrap the right-hand side over the left and press the two ends together so that the belt fastens comfortably. Press the ends of the garment against your body to ensure it is securely fastened.

Step 4. Then simply press the outer layer against the inner layer to fasten. (Fig. H)

5.3 Setting Up An Abdominal Muscle Treatment

Abdomen Garment Placement

Step 1: Position the abdomen garment around your waist so that the pads are pressed firmly against your skin, positioning the large square pad over your navel (belly button).

Step 2: Stretch the ends of the belt around your waist (Fig. I), until the small pads are between your hip bone and ribs on either side of your body (Fig. J). Fasten the belt tightly around your waist, but no so tight as to cause discomfort. The pads are water-based, so you may find they are cool when placed on your skin.

6.0 Using Garments with the Neurotech Plus for a Treatment

Prior to use, charge the battery. Insert your charger into an electrical outlet and then connect the Neurotech Plus to the charger (Fig. K). It may take approximately 2-3 hours to fully charge the battery. Your unit is fully charged when all three sections of the battery icon are full. The battery icon will flash when the battery is low and needs to be recharged.

NOTE: Do not leave your Neurotech Plus connected to the charger when the battery is fully charged.

Step 1: Connect the Neurotech Plus to the Conductive Garment. The Neurotech Plus must be connected to the Back or Abdomen Garment to work. Place a conductive garment on your back or abdomen as instructed. Plug the Neurotech Plus into the garment connector as shown. (Fig. L)

IMPORTANT: Make sure the Neurotech Plus and garment are connected correctly or your conductive garments will not work – this is a safety feature.

Step 2: Press and hold the on/off button (⏻) for three seconds to turn your Neurotech Plus on. (Fig. M)

Step 3: Select the program you wish to use (**P**) by pressing the program button. Program descriptions can be found in section 6.0 of Neurotech Plus manual or on page 17 of this manual. Ask your Health Care Professional for a recommendation on which program(s) to use.

Step 4: Press and hold the increase intensity buttons (Fig. N). Program descriptions can be found in section 6.0 of Neurotech Plus manual or on page 17 of this manual. Increase intensity using both arrows to a comfortable level. If you are using a TENS program for pain management, you will feel a slight tingling or tapping sensation. If you are using a NMES program, increase intensity until you feel your muscles contracting. Always choose an intensity level which is comfortable. This will vary from person to person. The increasing intensity level is displayed on the screen.

Step 5: Continue increasing the intensity throughout the session if possible. For NMES programs, you will feel the muscle contractions getting stronger as the intensity increases. Remember the harder you work your muscles the better the results you will see, but always reduce the intensity if you feel any discomfort. Your unit's intensity range is 0-99. You can utilize the keylock function when you find the best level of stimulation if you wish.

Pausing a Session

If you wish to pause a session before it is finished, simply press the on/off button briefly. The display will show that the unit is paused (**II**). To resume the session, briefly press the on/off button again. The display will return to its normal mode. (Fig. O)

Step 6: Session Over

At the end of the training session, your Neurotech Recovery - Back garments will stop automatically. However, to switch off your unit at any time during a session, press and hold the on/off button for two seconds. You should see the display turn off.

NOTE: Do not disconnect the Neurotech Plus from the garment without first switching the unit off, as this may result in an error on your display.

Step 7: Remove the garment by carefully peeling the garment with gel pads off your back or abdomen. Ensure that the gel pads remain attached to the garment. Place the garment on a flat surface with the gel pads facing upward and replace the liners on the black side of each pad (Fig. P). If the gel pads have lifted from the garment, press the edges back into position. Close the straps on the garment, ensuring that the gel pads all remain flat. Store the garments, with the gel pads still attached, on a flat surface in a cool dry place.

Congratulations on completing your first session!

Refer to Neurotech Plus manual for complete operating instructions.

7.0 System Maintenance & Storage

Caring for your Neurotech Recovery - Back garments

Washing Instructions

The conductive garments are washable but you must first remove the Neurotech Plus and conductive gel pads. Replace the liners on the **patterned** side of each gel pad until you are ready to use your Neurotech Recovery - Back garments again. You must always follow the instructions on the label when washing the garment. Never machine wash the Neurotech Recovery - Back garments. Hand wash in lukewarm water and NEVER wring to remove water. This protects the internal composition of the garment from damage.

Do not use any bleach when washing your Neurotech Recovery - Back garments.

Do not dry clean your Neurotech Recovery - Back garments.

Do not tumble dry your Neurotech Recovery - Back garments. The garments should be dried on a flat surface. Do not dry over anything hot (e.g. a radiator or any other direct heat source). Ensure the garment is completely dry before use. The straps should be fastened prior to washing.

The Neurotech Recovery - Back garments should not be ironed.

Storage

Remove the garment by carefully peeling the gel pads off of your back or abdomen. Ensure the gel pads remain attached to the garment. Place the garment on a flat surface with the gel pads facing upward and replace the liners on the gel pads. If the gel pads have lifted from the garment, press the edges back into position. Close the straps on the garment, ensuring the garment remains flat. Store the garment, with the gel pads still attached, on a flat surface in a cool dry place.

Repair, Service & Modification

Access to the interior is not required for maintenance purposes. If your Neurotech Recovery - Back garment is damaged, you should not use it, but should return it to the distributor that provided you with this product or Neurotech for repair or replacement. Repairs, service and modifications may not be carried out by anyone other than qualified service personnel authorized by Neurotech. Neurotech will not accept any responsibility if these guidelines and instructions are not followed.

8.0 Technical Information & Accessories

Neurotech Recovery - Back Gel Pads

Neurotech Recovery - Back uses gel pads which have been specifically designed for the back or abdomen garments. Replacement gel pads are available. To order replacement gel pads, please contact Neurotech or the Neurotech distributor that provided you with Neurotech Recovery - Back garments. Only gel pads specified by Bio-Medical Research Ltd. for use with Neurotech Recovery - Back may be used. All adhesive gel pads are provided with their own instructions where necessary, including cleaning, storage and care instructions. With proper care, as per the manual, gel pads have an estimated lifespan of 1 month.

Gel pads can lose adhesive strength. If any of the following occur causing loss of adhesive strength, we recommend you discontinue use of the gel pads and start with new gel pads:

- Build-up of oil, hair or skin on pad
- Dirt or other visible debris on pad
- Gel has removed from the pad
- General lack of adhesion - if after adding a couple drops of water gel is still not renewed

Note: Cleaning the surface of skin before use and replacing gel pad liners after treatments can help extend the use of gel pads.

Disposal of Gel Pads

Used gel pads must never be disposed of in a fire, but in accordance with your country's national laws governing the disposal of such items.

Product Type

Conductive Back Garment: Type 431	(1272-3301-01 - Small) (1272-3302-01 - Medium/Large)
Conductive Abs Garment: Type 437	(1271-3301)

Intended Use: Conductive Garment

Back Garment Materials

Main Panels: 89% Nylon, 11% Spandex
laminated to polyurethane. Silver trace.
Binding: 100% Cotton
Hook & Loop Fastener: 100% Nylon

Abdomen Garment Materials

Main Panels: 100% Nylon
Binding: 82% Nylon, 18% Elastane
Hook & Loop Fastener: 100% Nylon
Foam padding: 100% Polyurethane

Environmental Specifications

Operating Range:	Temperature: 10 to 35°C / 55 to 95°F
	Humidity: 20 to 65% RH
Storage Range:	Temperature: 0 to 55°C / 32 to 131°F
	Humidity: 10 to 90% RH

Description of the Device and Conductive Garment Symbols

There are a number of technical markings on your unit. These can be explained as follows:

This symbol means “Attention, consult the accompanying documents”.

European Union only: Indicates conformity to the requirements of the Medical Device Directive (93/42/EEC).

This symbol means “Read Instructions”.

The garment's batch number is represented on the garment packaging, by the number corresponding with the LOT symbol.

Disposal of Device

At the end of the product lifecycle, do not throw out this product with the normal household garbage, but bring it to a collection point for the recycling of electronic equipment.

Some product materials can be re-used if you bring them to a recycling point. By re-using some parts or raw materials from used products you can make an important contribution to the protection of the environment. Please contact your local authorities if you need more information about collection points in your area.

Waste Electrical and Electronic Equipment can have potentially harmful effects on the environment. Incorrect disposal can cause harmful toxins to build up in the air, water and soil and can be harmful to human health.

Accessories

Only gel pads specified by Neurotech, a division of Bio-Medical Research Ltd. may be used. Using other gel pads may degrade the performance levels.

Replacement Conductive Gel Pads

- Conductive Back Garment – Type 718, pack of 4
- Conductive Abs Garment – Type 709 x 1, 710 x 2 per pack

9.0 Warranty & Program Information

Should your conductive garment develop a fault within three years of purchase, Neurotech will undertake to replace or repair the conductive garment and parts found to be defective with no charge for labor or materials, provided the conductive garment*:

- has been used for its intended purpose and in the manner described in this instruction manual.
- has not been connected to an unsuitable power source.
- has not been subjected to misuse or neglect.
- has not been modified or repaired by anyone other than an approved Neurotech® agent.

This warranty complements existing national guarantee obligations and does not affect your statutory rights as a consumer.

*excludes conductive gel pads/electrodes.

Connecting to Neurotech Plus	Program Set # in Neurotech Plus Manual Section 6.0
Neurotech Recovery - Back Conductive Garment (Back garment)	1 (Type 431, U36)
Neurotech Recovery - Back Conductive Garment (Abdomen garment)	2 (Type 437, U37)
Lead wire Type 439, U41 for use with electrodes	4 (Type 439, U41)

Consult your physician for program set recommendation.

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About us:

neurotech® is a division of Bio-Medical Research (BMR) Ltd., a worldwide leader in providing innovative devices for pain relief and rehabilitative healthcare. Our 40 years of clinical experience and ongoing investment in scientific research and development has enabled us to deliver clinically proven solutions that are recommended and prescribed by leading health care professionals around the world.

Designed by & Manufactured for:

Bio-Medical Research Ltd., BMR House,
Parkmore Business Park West, Galway, Ireland.

Copyright © 2012 by Neurotech, all rights reserved. Neurotech Recovery - Back is designed by and manufactured for Neurotech. Neurotech is a registered trademark of BMR Ltd.

Part No.: 2401-3312

Issue Date: 3/13

Rev.: 6

Conductive Back Garment: Type 431

Conductive Abs Garment: Type 437